BG: ИНСТРУКЦИЯ ЗА УПОТРЕБА
EN: USER MANUAL
GR: ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
RO: INSTRUCTIUNI DE UTILIZARE
DE: GEBRAUCHSANWEISUNG
ES: INSTRUCCIONES DE USO
IT: ISTRUZIONI PER L'USO

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EN

IMPORTANT! READ CAREFULLY AND KEEP FOR FUTURE REFERENCE!

THESE INSTRUCTIONS CONTAIN IMPORTANT INFORMATION, GUIDELINES AND RECOMMENDATIONS FOR THE PRODUCT AND ITS SAFE USAGE.

Congratulations on your purchase of the Cangaroo electric breast milk pump. The unique all-in-one design allows mothers to conveniently collect breast milk no matter where they are. Depending on mothers' preferences and needs, it can be used as:

- **hands-free breast pump or**
- **a standard breast pump.**

The "hands-free" breast pump appliance has a user- comfortable design where you do not have to use your hands when expressing breast milk and can take it with you anywhere. Place the breast pump directly on your breast and put your bra back on. That way you can use it. You don't need to hold the breast pump with your hands close to you. The device can be used as a standard pump, providing exceptional comfort and gentle expressing of breast milk.

➤ **The Cangaroo electric breast pump provides many of the features which the busy mothers love and is perfect for many reasons:**

- Expressing and collecting breast milk while you are away from your baby.
- Relieving the tension in your breasts caused by the large amount of breast milk. For example, when your baby's feeding habits change because it has started eating more solids.
- Maintaining your milk supply if you have to temporarily stop breastfeeding.

➤ **The Cangaroo electric breast pump has the following features:**

➤ **Three modes in one machine.**

1. Stimulation mode – 9 Levels.
2. Suction mode – 9 levels.
3. Combination mode stimulation with suction – with 9 levels.

1. Use this product only for its intended purpose as described in this manual
2. This unit is not intended for usage by persons (including children) with reduced physical, sensory or mental capabilities or the lack of experience and knowledge unless they have been given supervision or instruction by a person responsible for their safety.
3. **Warning!** The breast pump is intended for repeated use only by one user.
4. **Warning!** Children should not play with the manual breast pump.
5. **Warning!** Before first usage check the breast pump, including the adaptor for signs of damage. If you find such, do not use the product until the fault is removed. Do not try to modify or repair the product by yourself. For consultation contact an authorised service or the sales agent, from which you have bought the manual breast pump.
6. Do not use any accessories, which are not approved by the manufacturer and do not make any improvements or changes on the product as this may lead to its malfunction or injuries to the person, who uses it.
7. Do not use spare parts, which are not the original ones supplied by the manufacturer or the importer. Using such, Your warranty card will be invalid.
8. **WARNING: To reduce the risk of electrocution, please:**
 - Do not use while bathing or showering.
 - Do not place or store this product where it can fall or be pushed into a bathtub or sink.
 - Do not place or drop into water or other liquid.
 - Do not reach after a product that has fallen into water. Unplug from the wall socket immediately.
 - Never operate this product if it has a damaged adapter or plug, if it is not working properly, if it has been dropped or damaged, or if it has been dropped into water.
 - Make sure the voltage of the AC adaptor is compatible with the power source.

Only use the AC adaptor which is included in the set. If any damage of the adaptor occurs, contact the manufacturer or the sales agent, from whom you have purchased the product.

 - The product should not be left unattended when plugged into the electrical wall socket.
 - Do not approach high temperatures or fire sources during the use of the pump.
9. Do not use while you are pregnant as this may lead to stimulating the birth process.
10. Never use while you are sleepy or drowsing.
11. Do not leave it and do not keep the product in a place with too high room temperatures or at direct sunlight.
12. Be careful not to drop the product and to put in its opening or in the feeding bottle other objects or liquids.
13. If you are infected with Hepatitis A, Hepatitis B or HIV (Human Immunodeficiency Virus), pumping breast milk does not reduce or remove the risk of transmitting the virus to your baby through breast milk.
14. Do not attempt to remove the pump body from your breast while under vacuum. Turn the unit off and break the seal between your breast and the silicone pad with your finger, then remove the pump from your breast.
15. Wash, rinse and sterilize all the parts of the breast pump after each use. Do not clean the part of the breast pump with strong detergents – alcohol-based ones, such containing bleach or with abrasive parts.
16. Never wash and sterilize the electrical parts of the product – the motor of the pump and the AC adaptor in order not to cause permanent damage to the product.
17. **WARNING!** This product is intended for personal usage, and it is not intended for the usage of other mothers.
18. The breastmilk bottle in the set is an integral part of the breast pump. For the purposes of this product use only this bottle and only for the breast milk collected with this pump.

II. PARTS DESCRIPTION AND CONTROL PANEL USE OF ELECTRIC PUMP

- **FIGURE PD.1:** 1.Silicone breast (nipple) shield– 1 pc.; 2.Silicone diaphragm – 1 pc.; 3.Pumping motor (mechanism) – 1 pc; 4.USB-C cable – 1 pc.; 5.Silicone valve – 1 pc.; 6.Milk container (180ml/ 6.00 oz) – 1 pc; 7.Bra adjuster; 8.Linker – 1 pc.
- **FIGURE CH.1:** Battery charging port when using the pump as "hands-free".
- **FIGURE PD.2:** 1.Silicone breast (nipple) shield– 1 pc.; 2.Silicone diaphragm – 1 pc.; 3. Pumping motor (mechanism) – 1 pc;

4. Silicone valve – 1 pc.; 5. Feeding bottle (180ml/ 6.00 oz) – 1 pc.; 6. Pump body (linker) – 1 pc.; 7. Dust cover – 1 pc.; 8. Horn mouth – 1 pc.

➤ **FIGURE CH.2:** Battery charging port when using the device as a standard pump.

➤ **FIGURE P:** a. **ON/OFF/PAUSE Button**; b. **Button +:** Level up; c. **Button -:** Level down; d. **Mode switch button/Light**; e. **Level indicator**; f. **Indicator showing session time in minutes**; g. **Battery charge indicator**.

III. CHARACTERISTICS

- **Battery capacity:** 3.7V -1000mAh
- **Rated input voltage:** 5V
- **Rated Input Current:** 1A
- **Working Power:** 5W
- **Charging time:** 120 - 180 minutes
- **Usage time after full charging:** 90 - 120 minutes
- **Expression power:** 150-270 mm/Hg
- **Stimulation power:** 75-195 mm/Hg
- **Shut time:** after 30 minutes of use
- **Noise limit:** ≤ 50 dB

IV. BATTERY CHARGE

1. Before first time use charge the battery completely.
2. The breast pump is with a built-in battery 3.7V/ 1000 mAh.

3. Charging:

- Connect the cable into the charging jack (see Figures CH.1 and CH.2).
 - Insert the plug into electrical outlet of the mains supply of your home.
 - We recommend you to leave the breast pump to charge completely before next use in order to prolong its battery life.
4. Use only the charger provided with the breast pump. Do not use chargers from other products.
 5. Charge the battery on time. Charge as soon as the battery indicator is with 1 bar or the indicator starts flashing.
 6. During charging, the battery indicator will blink. Once the battery is fully charged, the indicator will stop flashing.
 7. If you will not use the product for a long time, charge the battery completely before storing it. Then charge the battery at least once a month to prolong its life.
 8. If the battery is damaged or cannot be charged anymore, do not try to open the product by yourselves. For consultation and repair contact the shop from where you bought the product or the importer/ distributor.
 9. The battery is recyclable. If the product is damaged and cannot be used any longer, take it to an official collection point for electronic products' waste, batteries, chargers. DO NOT throw the product away with the household waste. Follow your country's rules for the separate collection of electrical and electronic products and rechargeable batteries. Dispose of waste of this type only at the designated places. This will be important and significant contribution for the environmental protection and the reduction of the harmful effects of the substances contained in batteries and electronic components on the environment and human health.

V. ASSEMBLY OF THE ELECTRIC BREAST PUMP

ATTENTION! NEVER STERILIZE, CLEAN WITH WATER OR SUBMERGE THE PUMP MECHANISM AND CHARGER IN WATER!

FIGURE A.1: BREAST PUMP „HANDS-FREE“

Follow the product assembly figures to ensure proper assembly.

1. **Step 1 (Figure 1):** Place the silicone valve at the lower end of the linker. Make sure it is inserted in the correct direction and fits tight around the ends of the linker. **Important!** If the valve is loose or not installed correctly, it will not be possible to create a sufficient vacuum !
2. **Step 2 (Figure 2):** Put the assembled linker into the milk container, with the “ear” part of the linker aligned with the internal slot of the milk container. Press the linker disc part with both hands to fasten the edges firmly to ensure complete sealing.
3. **Step 3 (Figure 3):** Install the silicone shield on the milk container. They should fit tightly together and there should be no gap between them.
4. **Step 4 (Figure 4):** Insert the silicone diaphragm into the upper part of the milk container. Make sure that it is properly fixed around the ends of the container opening.
5. **Step 5 (Figure 5):** Attach the pumping mechanism to the milk container. They should fit tightly together and there should be no gap between them.

FIGURE A.2: STANDARD BREAST PUMP

Follow exactly the instructions and sequence for assembling the product from the attached images and text.

1. **Step 1 (Figure 1):** Place the silicone valve at the lower end of the pump body. Make sure it is inserted in the correct direction and fits tight around the ends of the linker. **Important!** If the valve is loose or not installed correctly,

2.Step 2 (Figure 2): Attach the feeding bottle to the pump body by turning the bottle gently clockwise so that the thread of the bottle coincides exactly with the thread of the pump body.

3.Step 3 (Figure 3): Attach the horn mouth to the linker. Insert the silicone shield into the horn mouth and gently press them together to provide the necessary seal.

4.Step 4 (Figure 4): Insert the silicone diaphragm into the upper part of the pump body. Be careful not to tear or scratch the diaphragm because this will affect the pumping vacuum.

5.Step 5 (Figure 5): Insert the pumping mechanism and turn it clockwise.

VI. USE

- Wash your hands thoroughly and make sure your breasts are clean. Express some milk to check that your milk ducts are not blocked. The pump must be sterile and assembled.

- Sit down and relax in a comfortable chair, leaning forward slightly (Use a soft and comfortable pillow as a backrest).

➤**Placing the breast pump „hands-free“ design (Figure UM.1):** Put the pump on your breast. The silicone breast (nipple) shield should be firmly fitted against the breast, and the nipple should be centrally directed towards the breast pump opening. Keep the pump upright. Be careful not to move the pump because the loss of good contact with the breast will cause bad vacuum. Put your bra back over the pumping mechanism. Figure UM.1 shows the correct and incorrect positions to stand in while using the pump.

➤**Placing the standard breast pump (Figure UM.S):** Place the pump on your breast as shown in Figure UM.S.

➤**USE – FIGURE PU:**

1: Press on/off button to switch on the pump. Press the switch mode button to select one of the 3 functions of the pump.

- **Stimulation mode:** This mode imitates the natural way of sucking of the baby in the beginning of breast feeding, which is fast and light. Thus, the reflex for flow (secretion) of breast milk is stimulated and initiated.

- **Expression mode:** The mode mimics a baby's natural sucking with a high and gentle suction force to achieve an efficient result.

- **Combined mode stimulation with expression:** The mode combines stimulation with suction.

- When the mode selection button is pressed, the stimulation function is activated: 💧💧

- Repeatedly press the mode selection button to activate the expression function: 💧

- The next press of the mode selection button activates the combined mode stimulation with suction. 🍼

2: Each mode has 9 levels. With the "+" and "-" buttons you can adjust the levels of each function. **Note:** The pump will automatically shut off after 30 minutes of use. Press the on/off button to turn off the breast pump.

3.Memory function: Turn off the pump mechanism and the last used levels in massage mode and in suction mode will be automatically memorized.

➤ During use of the pump the level of the active mode will be shown on the display for 5 seconds and after that a timer will be displayed.

4. Once you have finished pumping, remove carefully the pump from your breast. Press the button on the front of the pumping mechanism and pull down the milk container. Hold the breast pump in an upright position and carefully remove the pump motor from the pump body (see Figure UM.2). Transfer the breast milk into a feeding bottle or milk storage bag.

5. Tips:

- If there is no result during pumping, do not continue! Try using the pump at another time of the day.

- On average, you have to pump for 8 minutes to drain 120 ml of milk. Keep in mind that the amount of milk varies from woman to woman.

- Do not overfill the milk bottle to prevent milk from leaking out of the bottle.

VII. WHEN TO USE YOUR BREAST PUMP

1. If it is possible. There should be a period of breastmilk secretion and breastfeeding before each use of the pump.

2. The electric pump is an apparatus, which helps postpartum women to ease the pain and to extend the breastfeeding time. It uses the premise that there is sufficient milk and smooth mammary gland. For some women, which are mothers for the first time, the mammary gland usually is not smooth enough. A reason sometimes can be the congenital nipple inversion or milk water shortage. So, we suggest you before purchasing and using the pump, to consult with a doctor who could confirm that your mammary gland is smooth and that the breast milk is in normal condition. Do not buy and use the pump only as a device to unblock your milk ducts.

VIII. HINTS

It takes practice in order to use the breast pump the best possible way. So, it will take you at least several attempts before starting successfully to use the electric breast pump.

1. Choose a quiet time to use the pump – time, when no one will disturb you.

2. Sit in a comfortable chair, in a perpendicular position to the ground. Please, put a comfortable cushion behind your back before you start using the breast pump. Take a glass of water and place it near you, because you may feel thirsty during the pumping process.

2. Sit in a comfortable chair, in a perpendicular position to the ground. Please, put a comfortable cushion behind your back before you start using the breast pump. Take a glass of water and place it near you, because you may feel thirsty during the

pumping process.

3. Place a warm cloth on your breasts for a few minutes before starting to express the milk as this may facilitate the process and relieve the tension in the breasts.
4. Warmth and relaxation can help for the easier pumping of the breast milk, so before using the pump, you might like to take a bath or a shower.
5. Try expressing from one breast while your baby is feeding from the other, or continue expressing just after the feed.

IX. HOW TO STORE THE BREAST MILK

1. Only breast milk that has been collected using a sterile breast pump can be stored to feed your baby.
2. The milk can be stored in a thermos at 40 °C, so you can feed your baby every 4 hours.
3. If you store freshly expressed milk, seal the milk. The milk can be stored for 4 hours at room temperature at 19 - 25 °C.
4. The expressed milk can be stored in a refrigerator for a maximum of 48 hours (but not on the refrigerator door).
5. The frozen breast milk (at temperature -18°C) can be stored up to 6 months. When deep freezing at -20°C, the breast milk can be stored for up to 12 months.
6. If there is some milk left in the bottle after feeding the baby, you can use this milk within a maximum of 2 hours.
7. Frozen milk must first be left in the refrigerator to thaw naturally. If you need it urgently, put it in a bowl of warm water and thaw it. After thawing, the milk can be stored in the refrigerator for 24 hours. **NEVER FREEZE MILK AGAIN.**
8. Frozen milk can be gradually warmed in warm water or placed in a baby bottle warmer.
9. **NEVER WARM YOUR BREAST IN THE MICROWAVE, BECAUSE YOU CAN DESTROY VITAL NUTRIENTS AND ANTIBODIES THIS WAY.** Also, uneven heating can lead to too hot areas of breast milk, which can cause burns to your baby's upper digestive tract.
10. Never place a frozen bottle of milk in boiling water, as this may cause the bottle to burst / break. Never pour boiling water directly into a feeding bottle. You should allow the water to cool for about 20 minutes before filling the bottle.

X. CLEANING AND STERILIZATION OF THE PUMP

- **Important: NEVER CLEAN THE PUMPING MECHANISM WITH WATER. You can only clean it with a soft dry cloth.**
- **Clean and sterilize your electric pump before first use and after each use, following the instructions below.**

❖ STERILIZATION:

1. **Disassemble the pump.** You can wipe up the pump motor (mechanism) with a slightly damp cloth and then you must dry it immediately.
2. Wash all other parts with warm soap water and rinse them well. Sterilize **ONLY** the following components of the pump: the silicone insert, the milk container, the milk bottle, the silicone valve, the silicone piston. The sterilization should not be more than 5 minutes.
3. The silicone valve and silicone insert are made of silicone and should not be sterilized for a long time under high temperature.

❖ RECOMMENDATIONS FOR CLEANING

1. Use a steam sterilizer. Boiling the parts is not recommended. The first reason is that the product will be easily damaged if improper sterilization is applied. Second, this method of sterilization cannot achieve full effect.
2. If you do decide to use boiling as a sterilization method, pour more water into the container so that the parts are completely covered with water. You must not leave the product unattended during the sterilization process. Do not boil for too long - maximum 10 minutes.
3. For breastfeeding mothers who breastfeed every 2-3 hours, we do not recommend sterilizing and cleaning after each use. Excessive disinfection and cleaning can easily lead to premature aging of parts and shorten the life of the breast pump.
4. Rinse the parts that come into contact with breast milk thoroughly after each use to prevent breast milk from drying on them. Thorough cleaning of stains and sterilization once a day is recommended.
5. Store the parts in a clean container until the next use.

• **IMPORTANT!** Before next use make sure that all parts of the breast pump are dry.

• If you use steam sterilizer to clean the pump, then be careful its part do not come into contact with the heating plate of the sterilizer. If you use boiling as a method of sterilization, do not allow the breast pump parts to come into contact with the bottom of the pan.

• If you use the breast pump as a means of storing milk, you must take care to disinfect all washable parts of the pump, because otherwise, the milk can easily spoil, as it is a product that is difficult to store.

XI. TROUBLESHOOTING GUIDE

1. Lack of suction:

- Check if you have assembled all parts correctly and if the silicone valve is positioned correctly.
- Ensure that the silicone shield brace is fitted securely to the pump body and a perfect seal is created.
- Check whether there are any damaged parts.
- If you can not turn the pump on, check whether the battery is exhausted. Connect the adapter to the terminal for charging the motor. Do not use it while charging as it shortens its battery life.
- Make sure that you have not blocked or covered the air opening of the pump body.

2. No milk is being expressed:

- Check point 1
- Ensure the pump is correctly assembled and that suction is being created.
- Relax and try again.
- We recommend you increase the levels gradually; the most suitable level is the one, with which the suction is painless. **The following is NOT true: the stronger – the better.** Too strong suction can hurt the nipple.

3. Pain in the breast area where there is expression:

- If you feel pain and no milk flows during expression, maybe your mammary gland is blocked. This case may result in mastitis. Contact a physician promptly. The complete milk expression will have good effect on mastitis protection.
- You may be using too high level of suction. You do not need to use all the suction that the pump can generate.
- Try to adjust the proper suction and frequency, starting at the first level and gradually increasing the levels.
- Before you start using it, put a warm cloth on your breasts and massage it for about 5 minutes.

4. Flow is too slow:

The speed of milk flow differs for every woman. The higher degree and level of expression do not mean faster flow of milk. This depends on many factors – constitution of the breast, level of possibility, even the mood.

• The more regularly you use the breast pump, the quicker and easier the flow of milk. Before you start using it, put a warm cloth on your breasts and use the massage mode for about 5 minutes before you switch to expression mode.

5. Milk is dropping down the silicone massage pad:

- Stop the pump immediately and clean it well.
- Remove and refit again the silicone cylinder in order to ensure it is firmly attached to the central part of the funnel.
- Try to lean slightly forward.

6. Cracking and discoloration of the pump/ feeding bottle parts:

Avoid contact with antibacterial or abrasive detergents/ cleaners as they can damage the plastic. Combination of detergents, cleaning products, sterilizing solution, softened water and temperature fluctuations can cause plastic to crack. If this occurs, please, do not use. The pump body and the breast milk containers are dishwasher safe but the food colorings may cause discoloring them. Keep the product out of direct sunlight as prolonged exposure may lead to discoloration.

7. Lost or broken parts:

For spare parts contact an authorized service center or the sales agent from whom you have purchased the product. Do not use spare parts, which are not approved or supplied by the manufacturer.

- Do not attempt to repair or to modify the product, because this may endanger Your and Your baby's health.

8. The pump does not work:

- The battery indicator will start blinking if the charge is not sufficient. Charge immediately.
- If even after charging the pump does not work, do not try to fix it yourself. Contact the facility from which you purchased it to have it repaired.

XII. CONFORMITY WITH EUROPEAN STANDARDS AND DIRECTIVES

This product has been designed and manufactured in accordance with the requirements of the European Directives: Directive 2014/35/EU relating to the making available on the market of electrical equipment designed for use within certain voltage limits; the Electromagnetic Coherence Directive 2014/30/EU; Directive 2011/65/EU and its supplements 2015/863/EU on the restriction of the use of certain hazardous substances in electrical and electronic equipment, Regulation (EU) 1935/2004 on materials and articles intended to come into contact with food; Regulation (EU) 10/2011 and its amendments (EU) 2018/213 and (EU) 2020/1245 on plastic materials and articles intended to come into contact with food and in particular the content of BPA (Bisphenol-A).

The product complies with the requirements of the following European standards, some of which are harmonised:

EN 60335-1:2012+A11:2014+A13:2017+A1:2019+A14:2019+A2:2019+A15:2021; EN 62233:2008; EN 14350:2020; EN IEC 55014-1:2021; EN 55014-2:2021; EN 61000-3-2:2014; IEC 62321-2:2021; IEC 62321-3-1:2013; IEC 62321-4:2013+AMD1:2017; IEC 62321-5:2013; IEC 62321-6:2015; IEC 62321-7-1:2015; IEC 62321-7-2:2017; IEC 62321-8:2017.

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EN WARRANTY: The product is guaranteed against any conformity defect in normal conditions of use as provided for by the instructions. The warranty shall not apply in the case of damage caused by improper use, wear or accidental events. For the duration of the warranty on conformity defects please refer to the specific provisions of applicable national standards in the country of purchase, where provided.

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