BG: ИНСТРУКЦИЯ ЗА УПОТРЕБА
EN: USER MANUAL
GR: ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
RO: INSTRUCȚIUNI DE UTILIZARE
ES: INSTRUCCIONES DE USO
HU: HASZNÁLATI UTASÍTÁS
CZ: NÁVOD K POUŽITÍ

NL: GEBRUIKERSHANDLEIDING
DE: GEBRAUCHSANWEISUNG
РУ: ИНСТРУКЦИЯ ПО ПРИМЕНЕНИЮ
IT: ISTRUZIONI PER L'USO
SR/MNE/BIH: UPUTSTVO ZA UPOTREBU
FR: NOTICE D'UTILISATION
SK: NÁVOD NA POUŽITIE


CELESTE DQ-S090


BG: ЕЛЕКТРИЧЕСКА ПОМПА ЗА КЪРМА
EN: ELECTRIC BREAST MILK PUMP
GR: ΗΛΕΚΤΡΙΚΟ ΘΗΛΑΣΤΡΟ
RO: POMPĂ ELECTRICĂ DE SÂN
ES: EXTRACTOR DE LECHE MATERNA ELÉCTRICO
IT: TIRALATTE ELETTRICO
DE: ELEKTRISCHE MILCHPUMPE
РУ: ЭЛЕКТРИЧЕСКИЙ МОЛОКООТСОС
NL: ELEKTRISCHE BORSTKOLF
SR/MNE/BIH: ELEKTRIČKA PUMPA ZA MAJČINO MLEKO
FR: TIRE-LAIT ÉLECTRIQUE
HU: ELEKTROMOS MELLSZÍVÓ
CZ: ELEKTRICKÁ ODSÁVAČKA MATEŘSKÉHO MLÉKA
SK: ELEKTRICKÁ ODSÁVAČKA MATERSKÉHO MLIEKA

PD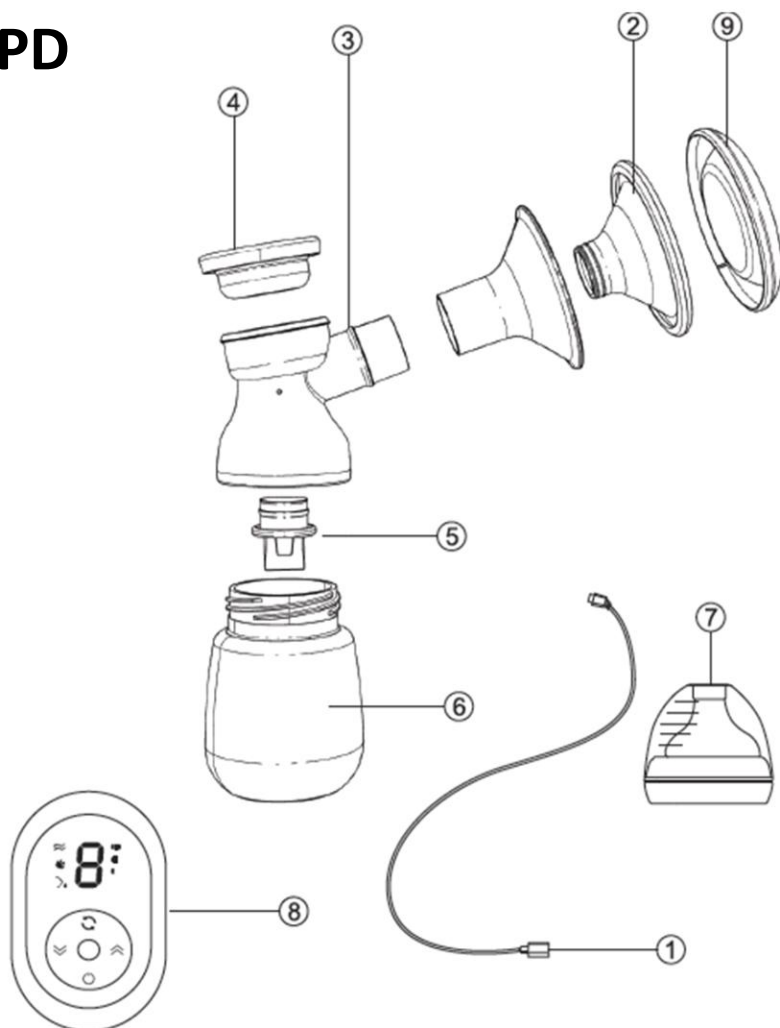


P

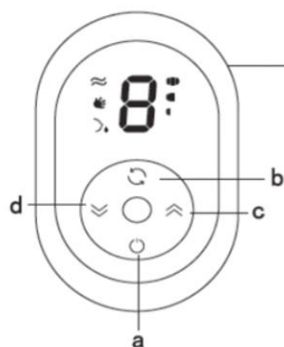a.ON/OFF


b.Change Mode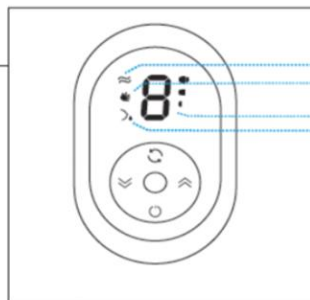

c.Increase Suction Level

d.Lower Suction Level

AD

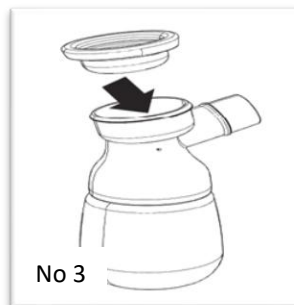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EN

IMPORTANT! READ CAREFULLY AND KEEP FOR FUTURE REFERENCE!

THESE INSTRUCTIONS CONTAIN IMPORTANT INFORMATION, GUIDELINES AND RECOMMENDATIONS FOR THE PRODUCT AND ITS SAFE USAGE.

Congratulations on the purchase of this electric pump. This product is an electric breast pump with different levels of control of the suction power, unique valve with gentle suction and soft massage silicone gel pad.

➤ **The Cangaroo electric breast pump provides many of the features which the busy mothers love and is perfect for many reasons:**

- Expressing and collecting breast milk while you are away from your baby.
- Relieving the tension in your breasts caused by the large amount of breast milk. For example, when your baby's feeding habits change because it has started eating more solids.
- Maintaining your milk supply if you have to temporarily stop breastfeeding.

➤ **The Cangaroo electric breast pump has the following features:**

1. Stimulation function – 9 Levels.
2. Suction function – 9 Levels.
3. Massage Function – 9 Levels.
4. Quiet motor providing discretion.
5. Light and easy to carry.
6. Easy to clean.

I. RECOMMENDATIONS AND WARNINGS FOR THE SAFE USAGE OF THE PRODUCT

1. Use this product only for its intended purpose as described in this manual
2. This unit is not intended for usage by persons (including children) with reduced physical, sensory or mental capabilities or the lack of experience and knowledge unless they have been given supervision or instruction by a person responsible for their safety.

3. Warning! The breast pump is intended for repeated use only by one user.

4. Warning! Children should not play with the manual breast pump.

5. Warning! Before first usage check the breast pump, including the adaptor for signs of damage. If you find such, do not use the product until the fault is removed. Do not try to modify or repair the product by yourself. For consultation contact an authorised service or the sales agent, from which you have bought the manual breast pump.

6. Do not use any accessories, which are not approved by the manufacturer and do not make any improvements or changes on the product as this may lead to its malfunction or injuries to the person, who uses it.

7. Do not use spare parts, which are not the original ones supplied by the manufacturer or the importer. Using such, Your warranty card will be invalid.

8. WARNING: To reduce the risk of electrocution, please:

- Do not use while bathing or showering.
- Do not place or store this product where it can fall or be pushed into a bathtub or sink.
- Do not place or drop into water or other liquid.
- Do not reach after a product that has fallen into water. Unplug from the wall socket immediately.
- Never operate this product if it has a damaged adapter or plug, if it is not working properly, if it has been dropped or damaged, or if it has been dropped into water.
- Make sure the voltage of the AC adaptor is compatible with the power source.
- Only use the AC adaptor which is included in the set. If any damage of the adaptor occurs, please, contact the manufacturer or the sales agent, from whom you have purchased the product.
- The product should not be left unattended when plugged into the electrical wall socket.
- Do not approach high temperatures or fire sources during the use of the pump.

9. Do not use while you are pregnant as this may lead to stimulating the birth process.

10. Never use while you are sleepy or drowsing.

11. Do not leave it and do not keep the product in a place with too high room temperatures or at direct sunlight.

12. Be careful not to drop the product and to put in its opening or in the feeding bottle other objects or liquids.

13. If you are infected with Hepatitis A, Hepatitis B or HIV (Human Immunodeficiency Virus), pumping breast milk does not reduce or remove the risk of transmitting the virus to your baby through breast milk.

14. Do not attempt to remove the pump body from your breast while under vacuum. Turn the unit off and break the seal between your breast and the silicone pad with your finger, then remove the pump from your breast.

15. Wash, rinse and sterilize all the parts of the breast pump after each use. Do not clean the part of the breast pump with strong detergents – alcohol-based ones, such containing bleach or with abrasive parts.

16. Never wash and sterilize the electrical parts of the product – the motor of the pump and the AC adaptor in order not to cause permanent damage to the product.

17. **WARNING!** This product is intended for personal usage, and it is not intended for the usage of other mothers.

18. The breastmilk bottle in the set is an integral part of the breast pump. For the purposes of this product use only this bottle and only for the breast milk collected with this pump.

FOR YOUR CHILD'S SAFETY AND HEALTH

WARNING!

- Always use this product with adult supervision.
- Never use feeding teats as a soother.
- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check food temperature before feeding.
- Keep components not in use out of the reach of children.
- Throw away at the first signs of damage or weakness.
- Inspect carefully before each use. Pull the soother in all directions. Throw away at the first signs of damage or weakness.

Do not leave the soother to direct sunlight or any other heat source, or in disinfectants (sterilizing solution) longer than recommended as this can weaken the teat.

II. PARTS DESCRIPTION AND CONTROL PANEL USE OF ELECTRIC PUMP

➤ **FIGURE PD - PARTS:** 1. Connecting tube + adapter for charging the battery of pumping mechanism 2. Silicone massaging pad; 3. Horn mouth and pump body; 4. Silicone Diaphragm; 5. Valve; 6. Feeding bottle; 7. Feeding teat and cap for the milk bottle; 8. Pumping mechanism ; 9. Dust shield.

➤ **FIGURE P – PANEL:**

A.Button ON/OFF

B.MODE SWITCH

C.Increase massage/suction level;

D.Reduce massage/suction level;

E.Function pumping mode display;

F.Function massage mode display;

G.Pumping level indicator;

H.Stimulation mode display.

III. ELECTRICAL CHARACTERISTICS

•**The supplied charger/adapter:** AC 100-240V 50/60Hz; DC: 5V/ 1 A

•**Built-in rechargeable battery:** 3.7V-1200 mAh

• **Power rating:** 4W

•**Noise limit:** ≤ 60dB(A)

•**Motor protection:** II

•**Conditions for use:** 5°C~ 40°C

•**Charging time:** 150-180 minutes

•**Use time after full charge:** 150 minutes

•**Method of control:** buttons

•**Display:** LED

IV. BATTERY CHARGE

1. Before first time use charge the battery completely.

2. The breast pump is with a built-in battery 3.7V/ 1200 mAh.

3. Charging:

-Connect the adapter to the charging jack of the pumping mechanism.

-Insert the plug into electrical outlet of the mains supply of your home.

-We recommend you leave the breast pump to charge completely before next use in order to prolong its battery life.

4. Use only the charger provided with the breast pump. Do not use chargers from other products.

5. Charge the battery on time. Charge as soon as the battery indicator is with 1 bar or the indicator starts flashing.

6. During charging, the battery indicator will blink. Once the battery is fully charged, the indicator will stop flashing.

7. If you will not use the product for a long time, charge the battery completely before storing it. Then charge the battery at least once a month to prolong its life.

8. If the battery is damaged or cannot be charged anymore, do not try to open the product by yourselves. For consultation and repair contact the shop from where you bought the product or the importer/ distributor.

9. The battery is recyclable. If the product is damaged and cannot be used any longer, take it to an official collection point for electronic products' waste, batteries, chargers. DO NOT throw the product away with the household waste. Follow your country's rules for the separate collection of electrical and electronic products and rechargeable batteries. Dispose of waste of this type only at the designated places. This will be important and significant contribution for the environmental protection and the reduction of the harmful effects of the substances contained in batteries and electronic components on the environment and human health.

V. ASSEMBLY OF THE ELECTRIC BREAST PUMP

ATTENTION! NEVER STERILIZE OR IMMEDIATE THE PUMP MECHANISM AND TUBE CONNECTING IT TO THE PUMP BODY. Follow the instructions and sequence for assembling the product, described in the text and images hereafter attached.

1. Step 1 (Figure 1): Place the valve (5) at the lower end of the pump body. Make sure that it is properly fixed around the ends of the pump body opening intended for it. **Important!** If the valve is loose or not installed correctly, it will not be possible to create a sufficient vacuum!

- 2. Step 2 (Figure 2):** Fix the feeding bottle (6) to the pump body by turning the bottle gently clockwise so that the thread of the bottle coincides exactly with the thread of the pump body.
- 3. Step 3 (Figure 3):** Insert the silicone diaphragm (4) into the upper part of the pump body. Be careful not to tear or scratch the cylinder because this will affect the pumping vacuum.
- 4. Step 4 (Figure 4):** Insert the pumping mechanism (8) and press it down to fix it.
- 5. Step 5 (Figure 5):** Insert the silicone massaging pad (2) into the horn mouth (3) and gently press them together to provide the necessary seal. Put the dust shield (9). Attach the assembled part to the pump body, making sure it is well fixed. The assembly is complete and you can use the breast pump.

VI. USE

1. Wash your hands thoroughly and make sure your breasts are clean. Express some milk to check that your milk ducts are not blocked. The pump must be sterile and assembled.
2. Sit down and relax in a comfortable chair, leaning forward slightly (Use a soft and comfortable pillow as a backrest).
3. Put the pump on your breast and start the pump motor.

4. Using the pump

A. Massage:

This feature stimulates the flow of breast milk. We recommend you using this option first and then switch to squeezing mode. The massage mode mimics baby sucking, not too intensive, and stimulates the flow of milk. The suction mode also mimics baby sucking, but is stronger than the first mode, in more frequent base and very efficiently.

Using the "▲" and "▼" buttons you can adjust the level you are most comfortable with for breast milk stimulation.. This mode has 9 levels. Once breast milk starts flowing, switch to suction mode.

B. Stimulation:

This mode imitates the natural way of sucking of the baby in the beginning of breast feeding, which is fast and light. Thus, the reflex for flow (secretion) of breast milk is stimulated and initiated. This mode has 9 levels. You can adjust the levels with the "▲" and "▼" buttons.

C. Suction:

Press the mode selection button and activate the suction function. Using the "▲" and "▼" buttons you can adjust the level of squeezing that is most efficient and comfortable for you. This mode has 9 levels.

5. On average, you have to pump for 10-15 minutes to drain 180 ml of milk. Keep in mind that the amount of milk varies from woman to woman.

6. Do not overfill the milk bottle to prevent milk from leaking out of the bottle.
7. When you stop using the pump, switch it off from the ON / OFF button. Then remove the tube from the back of the pump body. Remove the pump body from the bottle.
8. Battery charge indicator:
 - The battery charge indicator flashes when charging and stops when charging is complete.
 - When the battery is low, the indicator starts flashing and the pump automatically switches off.

VII. WHEN TO USE YOUR BREAST PUMP

- 1. If it is possible.** The use of the electric breast pump may begin usually two or four weeks after the birth of your baby. There should be a period of breast milk secretion and breastfeeding before each use of the pump.
2. The electric pump is an apparatus, which helps postpartum women to ease the pain and to extend the breastfeeding time. It uses the premise that there is sufficient milk and smooth mammary gland. For some women, which are mothers for the first time, the mammary gland usually is not smooth enough. A reason sometimes can be the congenital nipple inversion or milk water shortage. So, we suggest you before purchasing and using the pump, to consult with a doctor who could confirm that your mammary gland is smooth and that the breast milk is in normal condition. Do not buy and use the pump only as a device to unblock your milk ducts.

VIII. HINTS

1. It takes practice in order to use the breast pump the best possible way. So, it will take you at least several attempts before starting successfully to use the electric breast pump.
2. Choose a quiet time to use the pump – time, when no one will disturb you.
3. Sit in a comfortable chair, in a perpendicular position to the ground. Please, put a comfortable cushion behind your back before you start using the breast pump. Take a glass of water and place it near you, because you may feel thirsty during the pumping process.
4. Place a warm cloth on your breasts for a few minutes before starting to express the milk as this may facilitate the process and relieve the tension in the breasts.

5. Warmth and relaxation can help for the easier pumping of the breast milk, so before using the pump, you might like to take a bath or a shower.
6. Try expressing from one breast while your baby is feeding from the other, or continue expressing just after the feed.
7. Repositioning the pump on your breast from time to time can help stimulate the secretion. Practice in order to find the best pumping technique for you. If the process is causing you breast pain, stop immediately and consult with a doctor at once.

IX. HOW TO STORE THE BREAST MILK

1. Only breast milk that has been collected using a sterile breast pump can be stored to feed your baby.
2. The milk can be stored in a thermos at 40 ° C, so you can feed your baby every 4 hours.
3. If you store freshly expressed milk, seal the milk. The milk can be stored for 10 hours at room temperature at 19 - 25 ° C.
4. The expressed milk can be stored in a refrigerator for a maximum of 48 hours (but not on the refrigerator door) or in a freezer for a maximum of 3 months.
5. Frozen milk must first be left in the refrigerator to thaw naturally. If you need it urgently, put it in a bowl of warm water and thaw it. After thawing, the milk can be stored in the refrigerator for 24 hours.
- 6. NEVER FREEZE MILK AGAIN.**
7. Frozen milk can be gradually warmed in warm water or placed in a baby bottle warmer.
- 8. NEVER WARM YOUR BREAST IN THE MICROWAVE, BECAUSE YOU CAN DESTROY VITAL NUTRIENTS AND ANTIBODIES THIS WAY.** Also, uneven heating can lead to too hot areas of breast milk, which can cause burns to your baby's upper digestive tract. Never place a frozen bottle of milk in boiling water, as this may cause the bottle to burst / break. Never pour boiling water directly into a feeding bottle. You should allow the water to cool for about 20 minutes before filling the bottle.

X. CLEANING AND STERILIZATION OF THE PUMP

- 1. Important:** NEVER CLEAN WITH WATER or STERILIZE THE PUMPING MECHANISM, CONNECTING TUBE AND THE ADAPTER FOR CHARGING THE BATTERY. You can only clean it with a soft dry cloth.
2. Clean and sterilize your electric pump before first use and after each use, following the instructions below.
3. We recommend using an electric sterilizer, microwave sterilizer or boiling to clean the pump.
4. Do not use a UV sterilizer or dishwasher to clean the pump
- 5. STERILIZATION:**
 1. Remove the pump mechanism (D) and set it aside. Separate all other parts, wash them in warm soapy water and dry them. Do not use antibacterial or corrosive cleaners.
 2. Sterilize ONLY the following parts: Silicone massaging pad (2); Horn mouth and pump body (3); Silicone diaphragm (4); Valve (5); Feeding bottle (6); Nozzle for milk bottle (7); Dust shield (9).
- Maximum sterilization time in a steam or boiling sterilizer is about 8 to 10 minutes.
- 6. RECOMMENDATIONS FOR CLEANING**
 - Use a steam sterilizer. Boiling the parts is not recommended. The first reason is that the product will be easily damaged if improper sterilization is applied. Second, this method of sterilization cannot achieve full effect.
 - If you do decide to use boiling as a sterilization method, pour more water into the container so that the parts are completely covered with water. You must not leave the product unattended during the sterilization process. Do not boil for too long - maximum 10 minutes.
 - For breastfeeding mothers who breastfeed every 2-3 hours, we do not recommend sterilizing and cleaning after each use. Excessive disinfection and cleaning can easily lead to premature aging of parts and shorten the life of the breast pump. Thorough cleaning of stains and sterilization once a day is recommended.
 - Store the parts in a clean container until the next use.
 - **IMPORTANT!** If you use the breast pump as a means of storing milk, you must take care to disinfect all washable parts of the pump, because otherwise, the milk can easily spoil, as it is a product that is difficult to store.
 - **ATTENTION!** Before use, make sure that the pump body, valve, piston and cover are dry.
 - If you use a steam sterilizer to clean the pump parts; you must take care that they do not come into contact with the sterilizer heater. And if you use boiling as your sterilization method, be careful that the parts do not come into contact with the bottom of the container you are boiling in.
 - When not using the product for a long time, it is recommended to charge the battery once a month.

XI. TROUBLESHOOTING GUIDE

1. Lack of suction:

- Make sure that you have not blocked or covered the air opening of the pump body.
- Check if you have assembled all parts correctly and if the valve is positioned correctly.

- Ensure that the silicone massage pad is fitted securely to the pump body and a perfect seal is created.

- Check if you have placed the silicone valve firmly to the opening of the pump body.

- Check whether there are any damaged parts.

If you can not turn the pump on, check whether the battery is exhausted. Connect the adapter to the terminal for charging the motor. Do not use it while charging as it shortens battery life

2. No milk is being expressed:

- Check point 1

- Ensure the pump is correctly assembled and that suction is being created.

- Relax and try again.

- We recommend you increase the levels gradually; the most suitable level is the one, with which the suction is painless.

The following is NOT true: the stronger – the better. Too strong suction can hurt the nipple.

3. Pain in the breast area where there is expression:

- If you feel pain and no milk flows during expression, maybe your mammary gland is blocked.

The complete milk expression will have good effect on mastitis protection. You may be using too high level of suction. You do not need to use all the suction that the pump can generate.

- Try to adjust the proper suction and frequency.

4. Flow is too slow:

- The speed of milk flow differs for every woman. The higher degree and level of expression do not mean faster flow of milk. This depends on many factors – constitution of the breast, level of possibility, even the mood.

- The more regularly you use the breast pump, the quicker and easier the flow of milk. Before you start using it, put a warm cloth on your breasts and use the massage mode for about 5 minutes before you switch to expression mode.

5. Milk is dropping down the silicone massage pad:

- Remove and refit again the massage pad in order to ensure it is firmly attached to the central part of the funnel.

- Try to lean slightly forward.

6. Cracking and discoloration of the pump/ feeding bottle parts:

Avoid contact with antibacterial or abrasive detergents/ cleaners as they can damage the plastic. Combination of detergents, cleaning products, sterilizing solution, softened water and temperature fluctuations can cause plastic to crack. If this occurs, please, do not use. The pump body and the breast milk containers are dishwasher safe but the food colorings may cause discoloring them. Keep the product out of direct sunlight as prolonged exposure may lead to discoloration.

7. Lost or broken parts:

- For spare parts contact an authorized service center or the sales agent from whom you have purchased the product. Do not use spare parts, which are not approved or supplied by the manufacturer.

- Do not attempt to repair or to modify the product, because this may endanger Your and Your baby's health.

XII. CONFORMITY WITH EUROPEAN STANDARDS AND DIRECTIVES

This product has been designed and manufactured in accordance with the requirements of the European Directives: the Electromagnetic Coherence Directive 2014/30/EU; Directive 2011/65/EU and its supplements 2015/863/EU and 2017/2102/EU on the restriction of the use of certain hazardous substances in electrical and electronic equipment, Directive 2011/8/EU on the restriction of the use of bisphenol A in plastic infant feeding bottles; Regulation 10/2011/EU on plastic materials and articles intended to come into contact with food.

The product complies with the requirements of the following European standards, some of which are harmonised: EN 60335-1:2012+A11:2014+A13:2017+A1:2019+A14:2019+A2:2019+A15:2021; EN 62233:2008; EN 14350:2020; EN IEC 55014-1:2021; EN 55014-2:2021; EN 61000-3-2:2014; IEC 62321-2:2021; IEC 62321-3-1:2013; IEC 62321-4:2013+AMD1:2017; IEC 62321-5:2013; IEC 62321-6:2015; IEC 62321-7-1:2015; IEC 62321-7-2:2017; IEC 62321-8:2017.

Contents: PP and silicone; MADE FOR CANGAROO

Importer and Manufacturer: Moni Trade LTD

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EN WARRANTY: The product is guaranteed against any conformity defect in normal conditions of use as provided for by the instructions. The warranty shall not apply in the case of damage caused by improper use, wear or accidental events. For the duration of the warranty on conformity defects please refer to the specific provisions of applicable national standards in the country of purchase, where provided.

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